medi GmbH & Co. KG  
Medicusstraße 1  
D-95448 Bayreuth  
Germany  
T +49 921 912-0  
F +49 921 912-780  
ortho@medi.de  
www.medi.de

medi Australia Pty Ltd  
83 Fennell Street  
North Parramatta NSW 2151  
Australia  
T +61-2 9890 8696  
F +61-2 9890 8439  
sales@mediaustralia.com.au  
www.mediaustralia.com.au

medi Austria GmbH  
Adamgasse 16/7  
6020 Innsbruck  
Austria  
T +43 512 57 95 15  
F +43 512 57 95 15 45  
vertrieb@medi-austria.at  
www.medi-austria.at

medi Belgium bvba  
Posthoornstraat 13/1  
3582 Koersel  
Belgium  
T: + 32-11 24 25 60  
F: +32-11 24 25 64  
info@medibelgium.be  
www.medi-belgium.be

medi Brasil  
Rua Cristovan de Vita, n.º 260, Un. 30,  
Centro Logístico Raposo Tavares,  
Vila Camargo  
Vargem Grande Paulista - SP  
CEP 06730-000  
Brasil  
T: +55-11-3500 8005  
sac@medibrasil.com  
www.medibrasil.com

MAXIS a.s.,  
medi group company  
Slezská 2127/13  
120 00 Prague 2  
Czech Republic  
T: +420 571 633 510  
F: +420 571 616 271  
info@maxis-medica.com  
www.maxis-medica.com

medi Danmark ApS  
Vejlegardsvej 59  
2665 Vallensbaek Strand  
Denmark  
T +45-70 25 56 10  
F +45-70 25 56 20  
kundeservice@sw.dk  
www.meditandmark.dk

medi Bayreuth Espana SL  
C/Canigo 2-6 bajos  
Hospitalet de Llobregat  
08901 Barcelona  
Spain  
T +34-932 60 04 00  
F +34-932 60 23 14  
medi@mediespana.com  
www.mediespana.com

medi France  
Z.I. Charles de Gaulle  
25, rue Henri Farman  
93297 Tremblay en France Cedex  
France  
T +33-1 48 61 76 10  
F +33-1 49 63 33 05  
infos@medi-france.com  
www.medi-france.com

medi Hungary Kft.  
Bokor u. 21.  
1037 Budapest  
Hungary  
T +36 1 371-0090  
F +36 1 371-0091  
info@medi.hu  
www.medi.hu

medi Nederland BV  
Heusing 5  
4817 ZB Breda  
The Netherlands  
T +31-76 57 22 555  
F +31-76 57 22 565  
info@medi.nl  
www.medi.nl

# E+motion®

## Epicomed® E+motion®

Ellenbogenbandage  
Elbow support

Gebrauchsanweisung, Instructions for use, Mode d'emploi,  
Instrucciones de uso, Instrucoes para aplicacao, Istruzioni  
per l'uso, Navod k pouziti, Инструкция по использованию,  
Instrukcja zakładania.

medi Performance Collection

4 051526 930834

0013669/08.2018

medi. I feel better.

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#### Important notes

This medical device is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Do not wear the product over open wounds, and use it only as instructed by your doctor or orthotist.

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Polyamid, Polyester, Elasthan, Merino

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# Epicomed®

## Intended purpose

Epicomed is an elbow support. The product is exclusively to be used for the orthotic fitting of the elbow and only on unbroken skin or if any wounds have been properly covered.

## Areas of use

- Irritation at the tendon insertion (Epicondylitis humeri), e.g. tennis elbow
- Joint inflammation (arthritis)
- Painful joint wear (arthrosis)
- Irritation of the joint (e.g. excessive stress, injuries or operations) linked to swelling or bruising

## Side-Effects/Contraindications

No side-effect is known which influences the whole body. However, if bandages (or similar aids) are too tight, local pressure effects or constriction of blood vessels or nerves may occur in individuals. For this reason, you should discuss this with your doctor before applying the bandage if you suffer from:

- Diseases or injuries of the skin in the area of use, especially with inflammatory symptoms (excessive warming, swelling or reddening).
- Sensitivity and perfusion disorders of the legs or feet (e.g. in the case of diabetes).
- Lymph outflow disorders, as well as unclear swellings of soft tissues, apart from the area of use.

Any combination with other products (e.g. compression arm stockings) must be discussed in advance with the responsible doctor.

## Mode of Action/Comment on Wear

Epicomed causes a massage effect, which supports perfusion, by interacting with compressive knitted fabric and the two integrated silicon inlays. This effect furthers the decline in swellings and bruises. Irritation and pain are relieved. In addition, the lashing strap can be individually adjusted for the level of stress at work or during sport. This helps to reduce excessive stress on the musculature and tendons.

Supports cause the greatest effect during physical exercise. In principal, the support can be worn all day. However, this depends on how comfortable the support is to wear. You should avoid bending your arm tight for a long period. The support should be taken off for longer breaks (e.g. when sleeping).

If you suffer excessive pain or discomfort during wear, please take off the product immediately and contact your doctor or specialized shop.

The support is also suitable for use in water. After use, please rinse out the product thoroughly.

## Putting on/Taking off

- To make putting Epicomed on easier, fold down the upper fastening edge.
- Pull the bandage over the elbow so that the crease of the elbow sits in the middle of the knitted comfort zone and the seam on the back side runs over the middle of the elbow bone at the back of the arm. Then roll the upper part of the bandage back up. (Fig. 1-3)

- The position of the bandage is correct when the inner and outer bone (epicondyles) can be felt in the gaps in the silicone inserts. (Figs. 4 + 5)
- The strap enables the pressure to be regulated for each individual, depending on what their body requires (tightening, loosening). If required, the strap can also easily be removed whilst the device is being worn. (Fig. 6)

Please take care that trained staff instruct you when putting on the support for the first time.

---

### Care instructions

Velcro fasteners should be closed for washing. We recommend using a laundry bag. Soap residues, lotions and ointments can cause skin irritation and material wear. Fabric softeners, fats, oils, lotions, ointments and soap residues can affect the material, hinder the climate comfort effect and cause skin irritations.

- Wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

---

### Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

### Material composition

Polyamide, polyester, elastane, merino

### Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

### Disposal

The product can be disposed of in the domestic waste.

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