medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-780
ortho@medi.de
www.medi.de

medi Australia Pty Ltd
83 Fennell Street
North Parramatta NSW 2151
Australia
T +61-2 9890 8696
F +61-2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Belgium bvba
Posthoornstraat 13/1
3582 Koersel
Belgium
T: +32-11 24 25 60
F: +32-11 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Brasil
Rua Itapeva 538 cj. 42
Bela Vista, São Paulo - SP
CEP: 01332-000
Brazil
T +55-11-3500 8005
sac@medibrasil.com
www.medibrasil.com

MAXIS a.s.,
medi group company
Slezská 2127/13
120 00 Prague 2
Czech Republic
T: +420 571 633 510
F: +420 571 616 271
info@maxis-medica.com
www.maxis-medica.com

medi Danmark ApS
Vejlegardsvej 59
2665 Vallensbaek Strand
Denmark
T +45-70 25 56 10
F +45-70 25 56 20
kundeservice@sw.dk
www.medit danmark.dk

medi Bayreuth Espana SL
C/Canigo 2-6 bajos
Hospitalet de Llobregat
08901 Barcelona
Spain
T +34-932 60 04 00
F +34-932 60 23 14
medi@mediespana.com
www.mediespana.com

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33-1 48 61 76 10
F +33-1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Hungary Kft.
Bokor u. 21.
1037 Budapest
Hungary
T +36 1 371-0090
F +36 1 371-0091
info@medi.hu
www.medi.hu

medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31-76 57 22 555
F +31-76 57 22 565
info@medi.nl
www.medi.nl

medi Orient M.E.A. ZCC
TAURUS GROUP
Suite No. M01 Wasel Al Mamzar Bldg.
Al Wuhida Road, Al Mamzar
P.O. BOX NO. 91464
DUBAI, UAE
United Arab Emirates
T: +971-4 2557344
F: +971- 4 2557399
md@taurusgroup.net

medi Polska Sp. z o.o.
Zygmunta Starego 26
44-100 Gliwice
Poland
T +48-32 230 60 21
F +48-32 202 87 56
info@medi-polska.pl
www.medi-polska.pl

medi Bayreuth Unipessoal, Lda
Rua do Centro Cultural, no. 43
1700-106 Lisbon
Portugal
T +351-21 843 71 60
F +351-21 847 08 33
medi.portugal@medibayreuth.pt
www.medi.pt

medi RUS LLC
Business Center NEO GEO
Butlerova Street 17
117342 Moscow
Russia
T +7-495 374 04 56
F +7-495 374 04 56
info@medirus.ru
www.medirus.ru

medi Trading (Shanghai) Co. Ltd.
Unit 1806, Floor 18, Sandhill Plaza No. 1
2290 Zuchongzhi Road
200131 Shanghai
People's Republic of China
T: +86-1366 1668 687
F: +86-2161 7696 07
infocn@medi.cn

Medi Turk Ortopedi Medikal Ic ve Dis Tic. AS
Kültür mah. çaldıran sok.14/3
06420 Çankaya/Ankara
Turkey
T +90 312 435 20 26
F +90 312 434 22 82
info@medi-turk.com
www.medi-turk.com

medi UK Ltd.
Plough Lane
Hereford HR4 0EL
Great Britain
T +44-1432 37 35 00
F +44-1432 37 35 10
enquiries@mediuk.co.uk
www.mediuk.co.uk

medi

E+motion®

Genumedi®
E+motion®

Kniefbandage
Knee support

Gebrauchsanweisung, Instructions for use, Mode d'emploi,
Instrucciones de uso, Instruções para aplicação, Istruzioni
per l'uso, Navod k použití, Инструкция по использованию,
Instrukcja zakładania, Інструкція з використання, .
لیاورد، Návod na použitie, Instrucțiuni de utilizare, .
תארוך ושמיש.

medi Performance Collection

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Important notes

This medical device is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Do not wear the product over open wounds, and use it only as instructed by your doctor or orthotist.

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Polyamid, Elastan, Polyester, Merino

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Genumedi®

Intended purpose

Genumedi is a knee support with patella silicone ring. The product is exclusively to be used for the orthotic fitting of the knee and only on unbroken skin or if any wounds have been properly covered.

Areas of use

- Inflammation of the joint (Arthritis)
- Painful joint wear (arthrosis)
- Irritation of the joint (e.g. excessive stress, injuries, operations) linked to swelling and effusion
- Pain in the kneecap (femoropatellar pain syndrome, anterior knee pain)

Side-Effects / Contraindications

No general side effects are known. However, if supports (or similar aids) are too tight, local pressure effects or constriction of blood vessels or nerves may occur in individuals. If you have any of the following complaints please consult your doctor before using the brace:

- Diseases or injuries of the skin in the area of use, especially with inflammatory symptoms (excessive warming, swelling or reddening).
- Sensitivity and perfusion disorders of the legs or feet (e.g. in the case of diabetes).
- Lymph outflow disorders, as well as undiagnosed swellings of soft tissues, apart from the area of use.

Any combination with other products (e.g. compression stockings) must be discussed in advance with the responsible doctor.

Mode of Action / Comment on Wear

The integrated silicone ring of the Genumedi reduces stress on the kneecap. Together with the compression of the knitted fabric, it causes a massage effect. This enhances perfusion of the joint and decreases swelling and effusion. Irritation and pain are alleviated. At the same time, muscular control and inherent stabilisation of the knee joint are improved.

Supports cause the greatest effect during physical exercise. In principal, the support can be worn all day. However, this depends on how comfortable the support is to wear. Moreover, the knee may not be bent by more than 70° when sitting. The support should be removed for long breaks (e.g. long periods sitting, long drives or sleeping).

If you suffer excessive pain or discomfort during wear, please take off the product immediately and contact your doctor or specialised shop.

The support is suitable for use in water. After use, please rinse thoroughly

Putting on / Taking off

- Grasp the support with the thumb on the internal upper end of the lateral knob adhesive bands (Fig. 1)
- Pull the support over the knee in such a way that the silicone ring covers the kneecap centrally and free of pressure (Figs. 2 +3).
- To take off the support, grasp the lower lateral rods and pull it downwards and away (Fig. 4).

Please take care that trained staff instruct you when putting on the support for the first time.

Care instructions

The silicone ring is not supposed to be removed before washing. We recommend using a laundry bag. Fabric softener, fats, oils and ointments can attack the material and the Clima Comfort Effect can be impaired. Soap residues can cause skin irritation and material wear.

- Wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

**Storage instructions**

Keep the product in a cool, dry place and do not expose to direct sunlight.

Material composition

Polyamide, elastane, polyester, merino

Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

Disposal

The product can be disposed of in the domestic waste.

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