medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-783
ortho@medi.de
www.medi.de

medi Australia Pty Ltd
 83 Fennell Street
 North Parramatta NSW 2151
 Australia
 T +61-2 9890 8696
 F +61-2 9890 8439
 sales@mediaustralia.com.au
 www.mediaustralia.com.au

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
 T +43 512 57 95 15
 F +43 512 57 95 15 45
 verttrieb@medi-austria.at
 www.medi-austria.at

medi Belgium bvba
 Posthoornstraat 13/1
 3582 Koersel
 Belgium
 T: + 32-11 24 25 60
 F: +32-11 24 25 64
 info@medibelgium.be
 www.medibelgium.be

medi Brasil
 Rua Cristovan de Vita, n.º 260, Un. 30,
 Centro Logístico Raposo Tavares,
 Vila Camargo
 Vargem Grande Paulista - SP
 CEP 06730-000
 Brasil
 T: +55-11-3500 8005
 sac@medibrasil.com
 www.medibrasil.com

medi Canada Inc / médi Canada Inc
 597, Rue Duvernay, Verchères
 QC Canada J0L 2R0
 T: +1 450-583-3317 / +1 800 361 3153
 F: +1 888-583-6827
 service@medicanada.ca
 www.medicanada.ca

MAXIS a.s.,
 medi group company
 Slezská 2127/13
 120 00 Prague 2
 Czech Republic
 T: +420 571 633 510
 F: +420 571 616 271
 info@maxis-medica.com
 www.maxis-medica.com

medi Danmark ApS
 Vejlegardsvej 59
 2665 Vallensbaek Strand
 Denmark
 T +45-70 25 56 10
 F +45-70 25 56 20
 kundeservice@sw.dk
 www.meditandmark.dk

medi Bayreuth Espana SL
 C/Canigo 2-6 bajos
 Hospitalet de Llobregat
 08901 Barcelona
 Spain
 T +34-932 60 04 00
 F +34-932 60 23 14
 medi@mediespana.com
 www.mediespana.com

medi France
 Z.I. Charles de Gaulle
 25, rue Henri Farman
 93297 Tremblay en France Cedex
 France
 T +33-1 48 61 76 10
 F +33-1 49 63 33 05
 infos@medi-france.com
 www.medi-france.com

medi Hungary Kft.
 Bokor u. 21.
 1037 Budapest
 Hungary
 T +36 1 371-0090
 F +36 1 371-0091
 info@medi.hu
 www.medi.hu

medi Nederland BV
 Heusing 5
 4817 ZB Breda
 The Netherlands
 T +31-76 57 22 555
 F +31-76 57 22 565
 info@medi.nl
 www.medi.nl

medi Polska Sp. z o.o.
 Zygmunt Starego 26
 44-100 Gliwice
 Poland
 T +48-32 230 60 21
 F +48-32 202 87 56
 info@medi-polska.pl
 www.medi-polska.pl

medi Bayreuth Unipessoal, Lda
 Rua do Centro Cultural, no. 43
 1700-106 Lisbon
 Portugal
 T +351-21 843 71 60
 F +351-21 847 08 33
 medi.portugal@medibayreuth.pt
 www.medi.pt

medi RUS LLC
 Business Center NEO GEO
 Butlerova Street 17
 117342 Moscow
 Russia
 T +7-495 374 04 56
 F +7-495 374 04 56
 info@medirus.ru
 www.medirus.ru

medi Trading (Shanghai) Co. Ltd.
 Room 105, Building No. 4,
 Lane 500 ShengXia Road
 Pudong New District
 201210 Shanghai
 People's Republic of China
 T: +86-21 61761988
 F: +86-21 61769607
 info@medi.cn

Medi Turk Ortopedi Medikal Ic ve Dis Tic. AS
 Kültür mah. çaldıran sok.14/3
 06420 Çankaya/Ankara
 Turkey
 T +90 312 435 20 26
 F +90 312 434 22 82
 info@medi-turk.com
 www.medi-turk.com

medi UK Ltd.
 Plough Lane
 Hereford HR4 0EL
 Great Britain
 T +44-1432 37 35 00
 F +44-1432 37 35 10
 enquiries@mediuk.co.uk
 www.mediuk.co.uk

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Important notes

This medical device is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Do not wear the product over open wounds, and use it only as instructed by your doctor or orthotist.

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Lumbamed® plus

Intended purpose

The Lumbamed plus is a lumbar support with pad. The product is exclusively to be used for the orthotic fitting of the lower back and only on intact skin.

Areas of use

- Non-specific pain in the area of the lumbar spine (LS)
- (Repeated) Pain due to attrition in the area of the LS, e.g. on account of changes to the intervertebral disks
- Irritations of the sacroiliac joint
- Overstrain injuries in the transitional area lumbar spine/pelvis
- Weak or unbalanced trunk muscles (back, abdomen)

Side-Effects / Contraindications

No side-effect is known which influences the whole body. However, if supports (or similar aids) are too tight, this may lead to local pressure effects or the constriction of blood vessels or nerves in individual cases. You should thus consult your doctor in the following circumstances before applying the support:

- Diseases or injuries of the skin in the area of use, especially with inflammatory symptoms (excessive warming, swelling or reddening).

Any combination with other products must be discussed in advance with the responsible doctor.

Mode of action / Comment on wear

Lumbamed plus has a stabilising and relieving effect on the lumbar spine. Its compressive knitted fabric and the pad that is used also stimulate circulation and have a pain-relieving, massage effect without affecting the muscular force.

Supports have the greatest effect during physical exercise. In principal, the support can be worn all day. The special cut in the area of the groin (Comfort Zone) is decisive for the correct and comfortable fit of the support. However, this depends on how comfortable the support is to wear. The support should be taken off for longer breaks (e.g. sitting for a long time or a long car drive, sleeping).

If you suffer excessive pain or discomfort during wear, please take off the product immediately and contact your doctor or specialized shop.

The support is also suitable for use in water. After use, please rinse out the product thoroughly.

Putting on / Taking off

- Pull the textile cover over the massage pad and fix in the middle on the central, blue Velcro area between the two stays according to the instructions of your doctor or a specialist. (Fig. 1 + 2) The pad should be fastened so that its lower end rests just above the fold between the buttocks when worn.
- First slip your left hand into the internal wrist strap of the left side of the fastener. Guide the bandage round the body and then slip your right hand into the external hand strap of the right side of the fastener (Fig. 3 + 4)
- Pull both sides of the fastener into the middle so that they overlap as far as possible and stick to each other by the Velcro pads as evenly as possible. The bandage is properly fitted when the lower edge of the fastener is just above the groin (Fig. 5)

Please always wear the Lumbamed plus on the skin and under your clothing. This maximises the Clima Comfort Effect. Please take care that trained staff instruct you when putting on the support for the first time.

Care instructions

Velcro tabs must be closed for washing. Do not wash the pad with the support. It can be cleaned with a damp cloth. Fabric softener, fats, oils and ointments can attack the material and the Clima Comfort Effect can be impaired. Soap residues can cause skin irritation and material wear.

- You can wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

Material composition

Support: Polyamide, polyester, elastane, merino

Pad: Polyurethane, polyethylene

Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

Disposal

The product can be disposed of in the domestic waste.

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